



ECO™ II Cup Procedure Guide

Step 1:

Remove Eco Cup II from foil pouch.



Step 2:

Remove cup lid and provide urine specimen directly into the cup.



Step 3:

After urine specimen has been provided into the cup, Replace and secure the cup lid. Test will now start to run. Result will be complete in 5 minutes.




Step 4:

Peel off label to reveal the test result.



How to Interpret the Results

- Test is valid if there is a line next to “C”
- A line next to “T” = negative no matter how faint it is
- The absence of a line next to “T” = positive

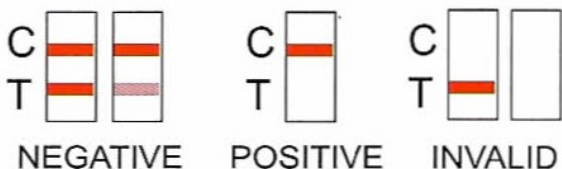

- THC = Marijuana
- COC = Cocaine
- AMP = Amphetamine
- MET = Methamphetamine
- OPI = Opiates
- BZO = Benzodiazepine

